

Fitting Guide



- Hold the pillows at your nose, ensuring the ResMed logo is at the top. Pull the headgear over your head.
- Headgear should sit midway between the ears and eyes.



- Adjust the top strap first.
- Pull the strap through the buckle one notch at a time.
- Do not overtighten.



- Adjust the back strap to fine-tune the fitting if necessary.
- Do not overtighten.
- The back strap can be positioned over or under the hair.



- Swift FX for Her pillows are designed to provide a natural fit.
- Check that the pillows are upright at the entrance to the nose.



- Connect free end of the short tube assembly to the device air tubing.
- Swift FX for Her is now fitted and ready for use.

Optional Soft Wraps



- Wrap each of the soft wraps around either side of the headgear, folding the top fastener over the lower one.
- The soft wraps should be positioned on your cheeks, with the fasteners on the outside.

Note: For additional information, refer to the Swift FX for Her Nasal Pillows System User Guide.

Swift™ FX for Her
NASAL PILLOWS SYSTEM